


# SV NAV+ Instructions

## Step 1


A. Once access to the vein is achieved, the user is ready to perform catheter preparation.



B. Remove PICC and cut it to length.

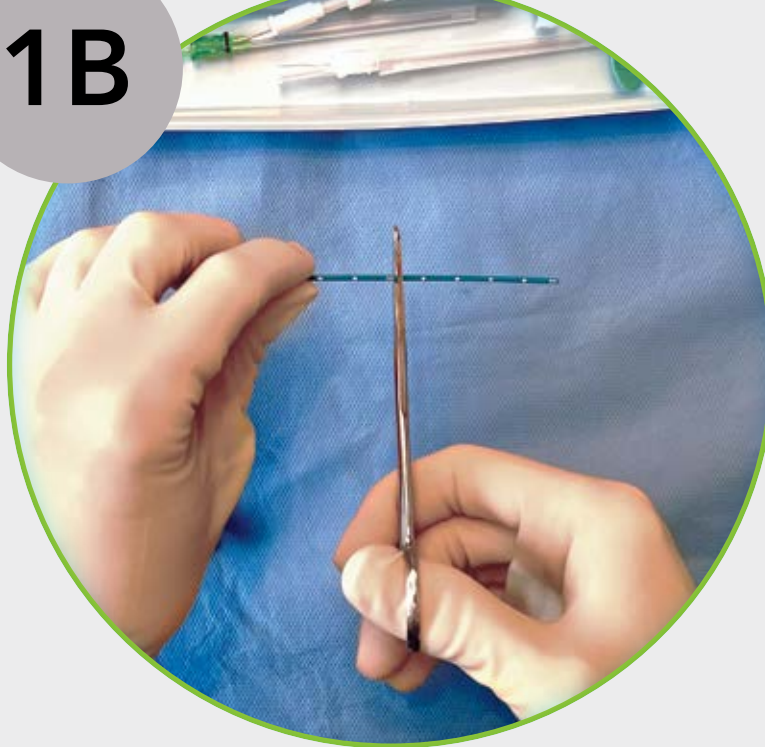


C. The lumens are then ready to be primed.

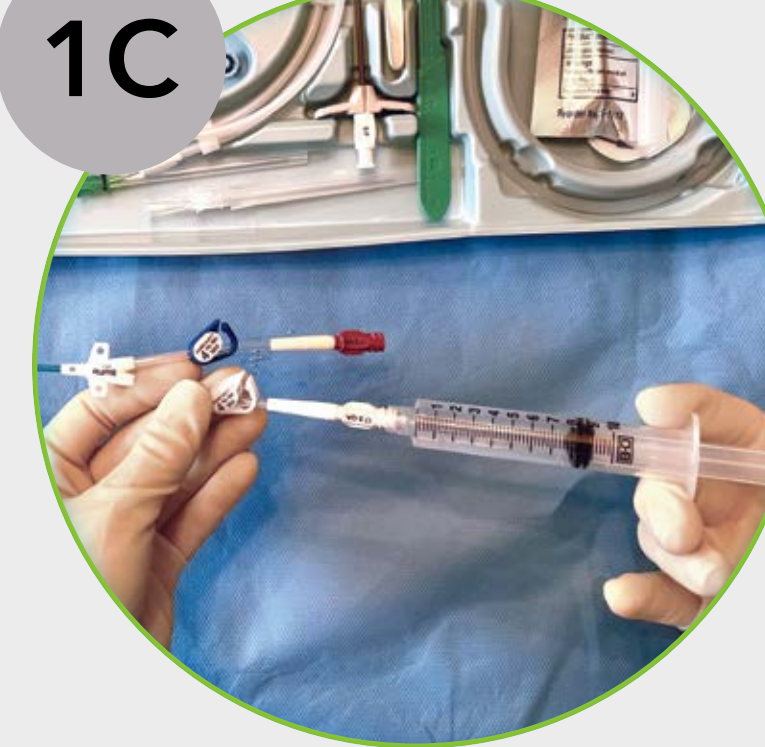

## Step 2

A. Load stylet into the PICC.

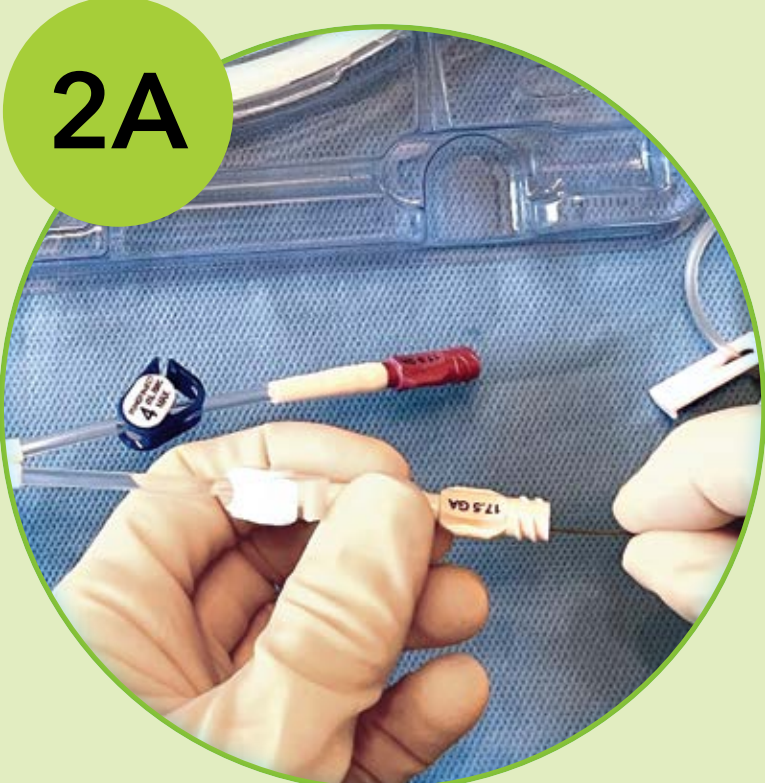

B. The stylet includes an atraumatic guiding tip which needs to be adjusted at the catheter hub until the guiding tip extends approximately 1.5 cm from the end of the PICC.




## Step 3

A. Fill the syringe with the 5% dextrose solution which is used to activate the blood flow sensors onboard the stylet.




B. Remove air from the syringe and connect syringe to the provided tubing set.

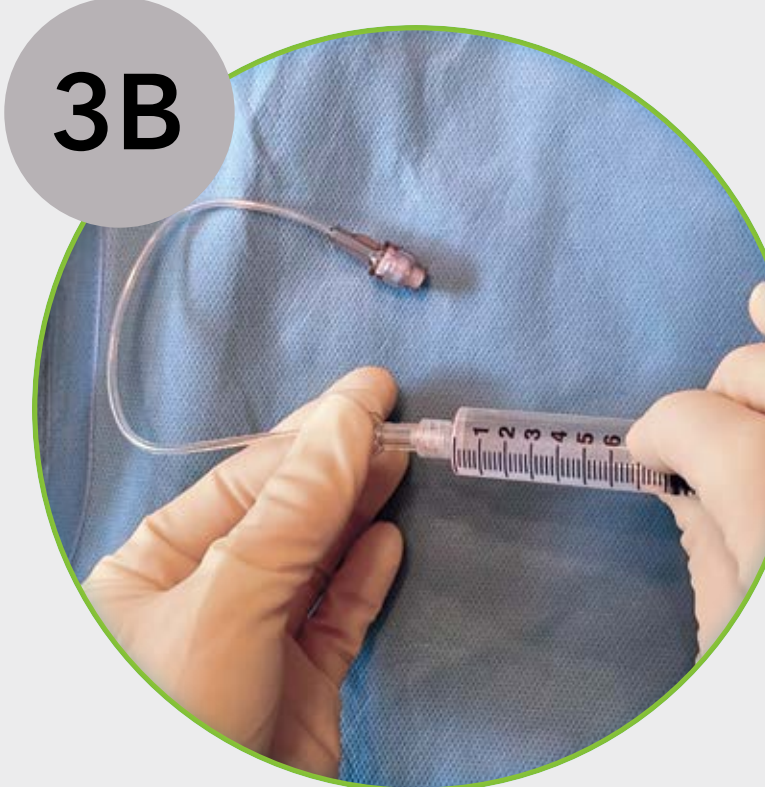


## Step 4


A. Load syringe into the sterile pump. The system is now pressurized and ready.

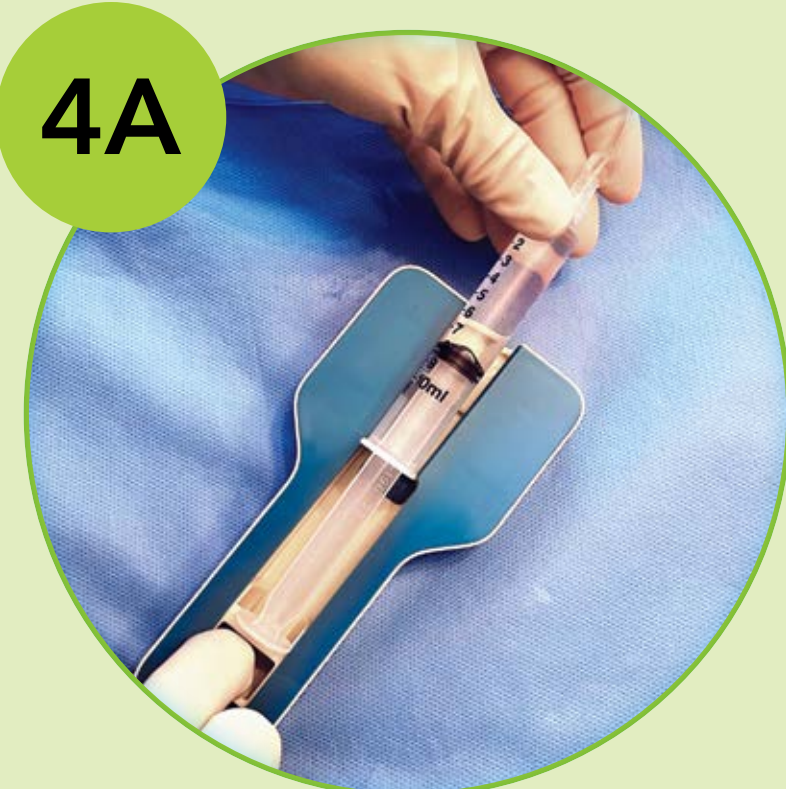

B. Connect tubing set to the stylet to begin the infusion of dextrose.

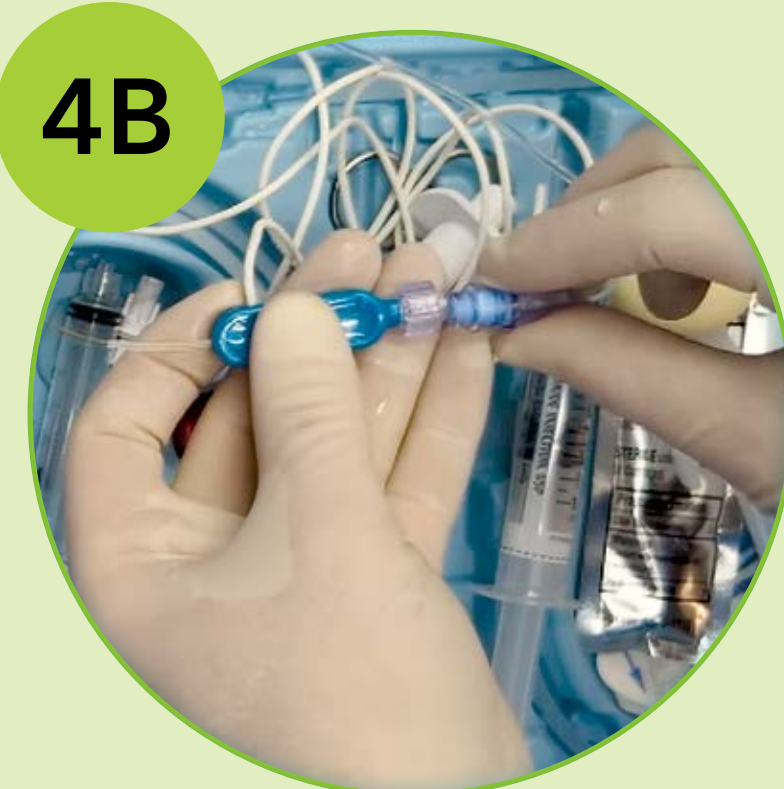

C. Once drops form on the tip of the stylet, connect to the system.

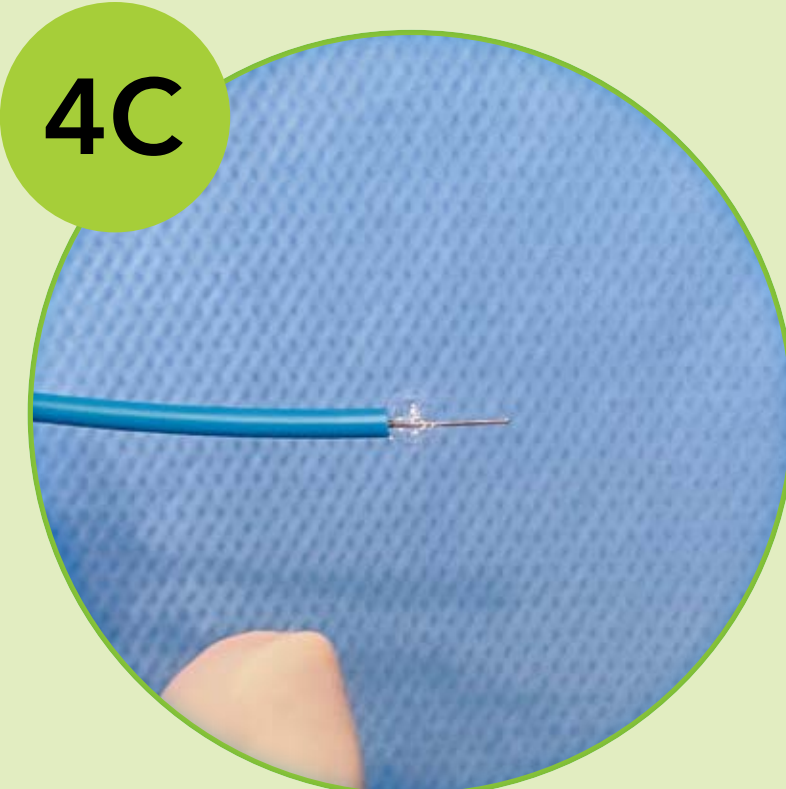

## Step 5

A. Insert PICC into venous sheath and advance until the tip is past the end of sheath.



B. Tap calibration screen and begin navigation.

