


C3 WAVE

CONNECT / CAPTURE / CONFIRM

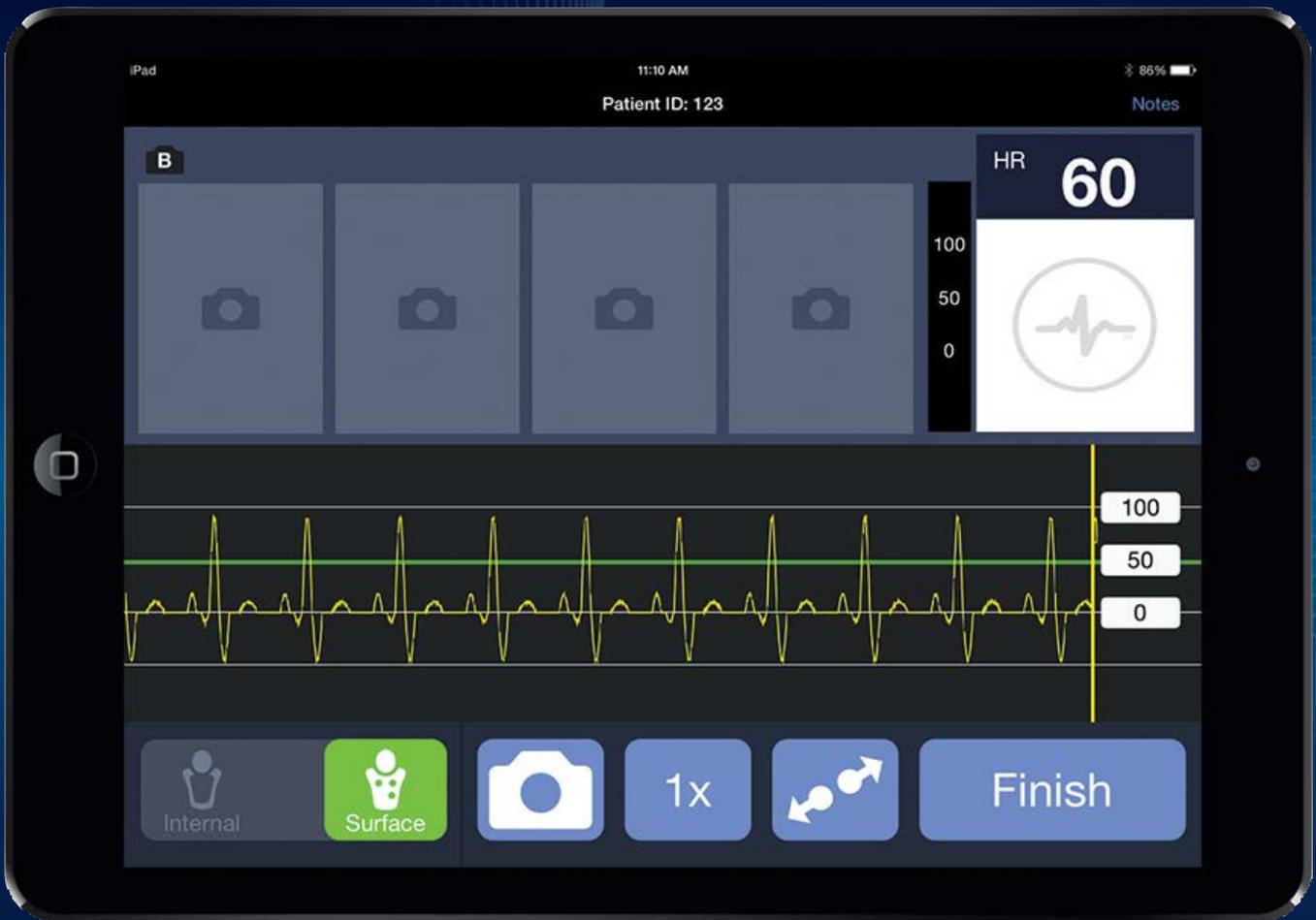

SPECTRUM
VASCULAR™

C3 WAVE

CONNECT / CAPTURE / CONFIRM



We are proud to introduce you to the next generation of ECG PICC placement with the C3 Wave System. The C3 Wave device is an easy to use, app-based wireless ECG system designed to free up clinicians hands by removing excess wires and allowing more focus on patient care.

FEATURES & BENEFITS

- ▶ Easy and intuitive from the very beginning
- ▶ Easily correlate catheter depth with P wave activity
- ▶ Seamlessly transition between extracavitary and intracavitary waveforms
- ▶ Intra-procedural system settings enable a stable and clear waveform
- ▶ Capture & compare multiple snapshots

Featuring advanced, yet easy-to-use, software - **the ECG waveform is wirelessly transmitted to a tablet** which allows the operator to view and record changes to the ECG waveform as the tip of the catheter approaches the heart. As the PICC approaches the atrium of the heart, the P wave in the ECG waveform shows substantial changes. This system is designed to aid the visualization of changes in P wave amplitude.

The C3 Wave System integration offers intelligent design and functionality to meet clinician needs. The charging base not only charges the patient hub, but also allows the user to store all wireless components in one location. Depending on preference, the charging base acts as a table top, or mounts to a pole. The two front spools allow the user to wrap the ECG leads when not in use. The remote is propped up to allow the user to easily place it in its sterile cover.



The use of Bluetooth technology eliminates the need to have cables running from the patient to the monitor.

Ordering Information

Capital System		
Catalogue Number	Description	Quantity
H787MRC3S02US0	C3 Wave System w/out iPad- United States	1/box

C3 Wave System Includes: (1) C3 Manual (1) Hub (1) Charger Base (1) Remote (1) Power Supply (1) ECG Snap Leads

Procedure Disposables		
Catalogue Number	Description	Quantity
H787MRC3RD0001	ECG Cable Accessory Pack (ECG Clip Cable, Sterile Remote Cover and Window Drape)	12/box
H787MRC3RD0031	ECG Cable Accessory Pack (ECG Clip Cable and Sterile Remote Cover)	25/box
H787MRC3RD0041	ECG Cable Accessory Pack (ECG Clip Cable, Sterile Remote Cover and Full Body Drape)	5/box

C3 Wave Accessories		
Catalogue Number	Description	Quantity
H787MRC3S900	C3 Wave System Hub	1/box
H787MRC3S910	C3 Wave System Remote	1/box
H787MRC3S920	C3 Wave System Hub Battery	1/box
H787MRC3S930	Brother Label Printer	1/box
H787MRC3S960	C3 Wave System Hub Charging Base	1/box
H787MRC3S970	C3 Wave System Remote Battery	1/box
H787MRC3S9160	C3 Wave System Pole Mount Kit	1/box
H787MRC3S940	C3 Wave System Pole Release Clamp	1/box
H787MRC3S9190	Hub Power Supply - Australia, New Zealand	1/box
H787MRC3S9200	Hub Power Supply - US	1/box
H787MRC3S9210	C3 Wave System ECG Lead Set	1/box



Consult your Spectrum Vascular representative for country specific product availability.

IMPORTANT RISK INFORMATION

INTENDED USE/ INDICATIONS FOR USE: The C3 Wave System is indicated for use in the positioning of Peripherally Inserted Central Catheters (PICC). The C3 Wave provides real-time catheter tip location information by displaying changes in the patient's cardiac electrical activity. The C3 Wave is indicated for use as an alternative method to chest X-ray or fluoroscopy confirmation of PICC tip placement in adult patients.

NOTE: Limiting, but not contraindicated, situations for this technique are patients where cardiac rhythms may change presentation of the P-Wave:

- Atrial fibrillation
- Atrial flutter
- Severe tachycardia
- Pacemaker-driven rhythm
- Chronic obstructive pulmonary disorder (COPD)

Such patients are easily identified prior to PICC insertion. Use of an additional confirmation method is necessary to confirm catheter tip location.

Refer to Directions for Use and/or User Manual provided with the product for complete Instructions, Warnings, Precautions, Possible Adverse Effects and Contraindications. Observe all instructions for use prior to use. Failure to do so may result in patient complications.

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.

www.spectrumvascular.com